

PRODIGY

MATTRESS OVERLAY[®]

The PRODIGY Mattress System is an adjustable, zoned product that offers you the ability to adjust the system to fit your individual needs. The objective is to "immerse" the user into the mattress system approximately 1 ½" from the base.

1. Turn valve counter-clockwise to open.
2. Place hand pump over valve and inflate until section arches slightly.
3. Remove pump hose from valve and turn clockwise to close.
Repeat steps for each mattress section.
4. Snap mattress sections together.
5. Place individual on mattress, preferably in a side-lying position.
6. Proceeding one section at a time, starting with the middle section, place your hand between the PRODIGY and the individual's deepest bony prominence.
7. With your hand in place, slowly turn the valve counter-clockwise to release some air to immerse the individual in the mattress system. When you can feel the base of the mattress with your finger tips (approx. 1 1/2" or 4cm) between the individual's deepest bony prominence and the mattress base, pinch the valve stem and turn the valve clockwise to close. Repeat for remaining sections.

Cleaning: Soap and warm water may be used to clean the PRODIGY Mattress. Rinse thoroughly and allow to dry. (Make sure valves are closed prior to immersing in water.) Do not use bleach when cleaning this product.

Repair: Two pressure adhesive patches are included with the PRODIGY Mattress System. If leak occurs, locate leak and clean surrounding area. Let surface dry. Peel backing from the patch provided and center over hole. Press firmly until a good seal is achieved. This product is now ready for use. Follow adjustment instructions above.

All returns require authorization from The ROHO Group and are subject to a restocking charge.

If you have any questions regarding the Prodigy Mattress Overlay, please contact The ROHO Group Customer Service Department at (800) 851-3449.

